

PULI JET PLUS

INSTRUCTIONS FOR USE

FOR CLEANING AND DISINFECTING

Puli Jet Plus is a non-foaming concentrate for disinfecting and simultaneous deodorization, cleaning and maintenance of dental aspirators and amalgam separators (wet, semi-wet & dry systems) and does not contain aldehydes and phenol.

1 litre of Puli Jet Plus makes up to 125 litres of disinfectant or 250 litres of cleaner.

Please always refer to the manufacturers instructions when using Puli Jet Plus

1. Unscrew red cap slightly.
2. Squeeze bottle to distribute liquid to top container.
3. To mix, fill the 'Pulse Cleaner Bucket' with warm water (approx temp 50°C) up to the third marker, then pour the Puli Jet Plus into the 'Pulse Cleaner bucket'.



General Cleaning: We recommend that you cleanse the aspiration system every evening, at the end of work. For this mix 10ml of Puli Jet Plus with 2.5 litres of warm water.

Disinfecting: We recommend to disinfect the aspiration system once a week with a higher concentrate of Puli Jet Plus. For this mix 20ml of Puli Jet with 2.5 litres of warm water.



HOW IS IT WE LEAD IN OUR FIELD, WHEN WE COST LESS THAN THE ALTERNATIVES? THIS IS HOW:

Constant research: this lets us apply the latest technology in all of our products and solutions.

We enhance performance: electronic and information technology allow us to increase the performance and reliability of our products.

We reduce costs: less maintenance and energy costs mean on a cost-benefit analysis we are always the most economical.

We reduce environmental impact: we save 50% of primary materials, and allow you to save between 30% and 50% of electrical consumption.



UK